


USER MANUAL



PRODUCT DESCRIPTION


Through the scanning of the full face, it can diagnose the face and visually analyse with enlarged images under spectrum. With the medical optical algorithm analysis, it will delivery medical skin analysis report and offer a set of diagnosis solution.

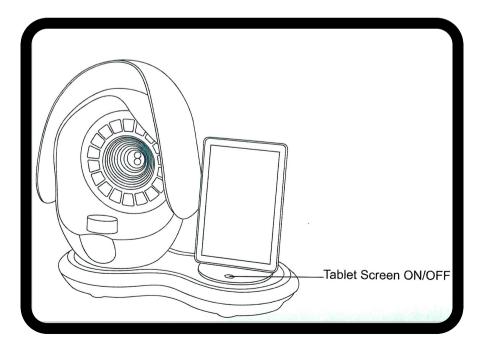



USE METHOD

- 1. Connect the power supply and open the power button on the back of the machine;
- 2. Press and hold the screen button for 3 seconds, wait for the system to start up, and automatically enter the system.
- 3. Make sure the internet connected.Connect the WIFI on the top of the screen.
- 4. Fill the complete customer information then login.
- 5. Go to the scanning page.
- 6. Lamp on,customer head straight in and Chin touch the steel.
- 7. Put down the black cover, then press scan button on the screen.
- 8. After the scanning is completed, open the black cover and the system auto into the analysis report.
- 9. Select the product to delivery with the report.
- 10. Check the report on the phone.

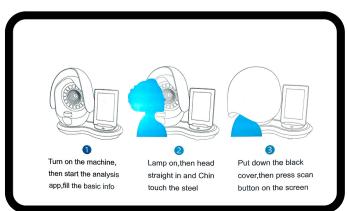




FUNCTIONAL EFFECT

FEATURES:

Digital camera pixels	:
Dual cameras	:
Face Al	:
Analysis items	:
Precise analysis result	:
Product recommendation :	
Huge database	:

20 million Pixels in HD camera Colorful and monochrome dual camera Intelligent positioning the full face Detection for 10 main skin problems Accurate analysis result att5% error rate Refer to the analysis result by build-in the platform Linking to the skin database if connect the internet

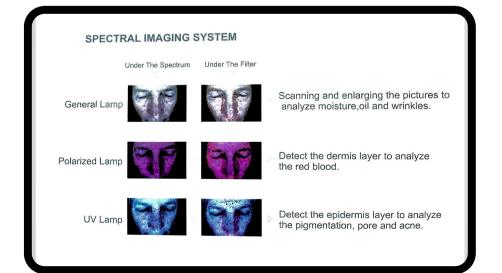



PRODUCT SPECIFICATIONS

Materials-	ABS+Stainless Steel	
Power-	30W	
Related Voltage-	AC 100-265V 50-60Hz	
Temperature -	-5 C~60 C	
Net Weight-	Approximate 7.5 KG	
Gross Weight-	Approximate 8.5 KG	
Size -	Approximate	
595mmx480mm×340mm		